


Let SOH-NAA
guide your way!

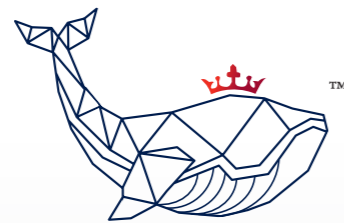

SOH-NAA **ULTRASOUND** **STANDOFF GEL PAD**

- FLEXIBLE
- LATEX FREE
- HYPOALLERGENIC

© Patent pending ARTG 386 392

ABN 71 638 909 994 www.ultrasoundgelpad.com contact@ultrasoundgelpad.com

Manufactured by: Fresco International 2005 SA, C/ Torre de Cellers, 5 08150 Parets del Vallés, Barcelona for sponsor Moby Holdings Pty Ltd

Ultrasound Standoff Gel Pad

The ultimate adjunct aid to diagnostic ultrasound.

- Flexible probe tissue interface
- Highest quality superficial tissue imaging
- Lossless acoustic transmission
- Application across all settings and tissue types
- Latex Free
- Hypoallergenic



Want to know more about our product Or ready to order?

Scan the QRCode today.



Gel Pad Properties

- Non-Sterile
- Latex free
- Flexible
- Medical Grade
- Polymer Mineral Oil Gel Pad
- Hypoallergenic

BEFORE USE



WASH




DRY



APPLY

HOW TO USE

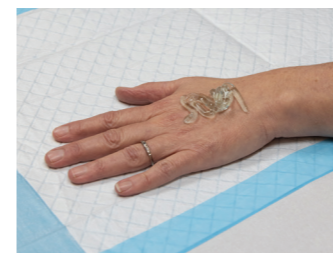

Step 1

Place acoustic gel on patient area of concern to provide couplant between patient and acoustic standoff pad



Step 2

Apply acoustic standoff pad onto gel



Step 3

Place gel on pad to provide couplant between acoustic standoff pad and transducer



Step 4

Scan Image

DO NOT USE



On broken skin surfaces



If pad is cracked or damaged in any way



If expired, replace as necessary